



# Product Usage Card

## M5 Bariatric Lift-Recline Chair

Applies to: M5 650 (320kg weight capacity)  
and M5 850 (320kg weight capacity).



Do not sit on the  
footrest



Assist footrest by  
lifting feet



Do not drop into chair  
when partially raised




Fully raise seat before  
sitting

- Do not exceed the M5 650's 320kg, and the M5 850's 320kg, weight capacity.
- Do not place anything over power cords, pass any power cords through the chair or drag the chair by power cords.
- Do not allow children to play on the mechanised furniture. Ensure all persons are clear of the chair when operating.
- The handset is water resistant only and can be wiped over but should not be immersed in liquid.
- The occupant must ease themselves into the chair in the raised position. Slumping or dropping into the chair may damage the mechanism or motor and void the warranty.
- Follow the fabric care instructions label provided with the chair.

### Linak Motor and Hand Control

**Charge the battery for 24 hours prior to first use.**

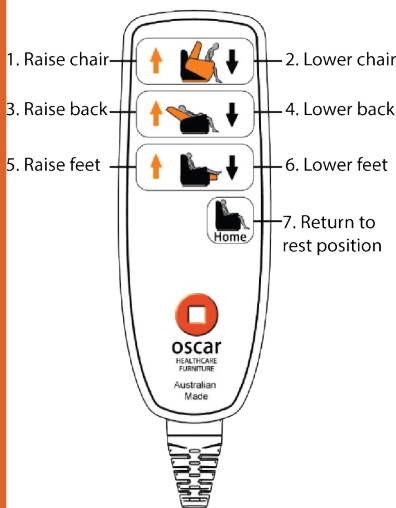

**Specifications:** The supply voltage is 240 volts AC, 50Hz. A battery charger external to the motor reduces the voltage entering the chair to 24 volts DC. Both motors and the hand control operate at 24 volts DC.

The calibration of motors and functions in your lift chair are pre-set from the factory and should not require adjustment for the life of the chair. In the event of a service issue, contact Oscar Furniture. When the chair beeps, re-charge for 24 hours. Do not allow the battery to run flat. The fully charged battery can provide approximately 30 cycles of use.

**For assistance to sit:** Half recline the back (4); raise the seat (1); lean into the seat; lower the chair (2); Home (7) returns to rest position.

**For assistance to stand:** Half recline back (4); raise the seat (1); stand away from the chair. Home (7) returns to rest position.