



# Product Usage Card

## Standard Lift and Recline Chair

Applies to: Barwon, Hudson, Cloud, Menningham (superseded), and Brumby (superseded).  
130kg weight capacity



Do not sit on the footrest



Assist footrest by lifting feet



Do not drop into chair when partially raised



Fully raise seat before sitting

- Do not exceed the chair's 130kg weight capacity.
- Do not place anything over power cords, pass any power cords through the chair or drag the chair by power cords.
- Do not allow children to play on the mechanised furniture.
- Ensure all persons are clear of the chair when operating.
- The handset is water resistant only and can be wiped over but should not be immersed in liquid.
- The occupant must ease themselves into the chair in the raised position. Slumping or dropping into the chair may damage the mechanism or motor and void the warranty.
- Follow the fabric care instructions label provided with the chair.

### Linak Motor and Hand Control

Raise backrest,  
Close footrest,  
Raise chair.

Lower backrest,  
Open footrest,  
Lower chair.

Emergency lower,  
Return to the upright  
seated position.



**Specifications:** The supply voltage is 240 volts AC, 50Hz. A transformer external to the motor reduces the voltage entering the chair to 24 volts DC. Both motors and the hand control operate at 24 volts DC.

**Maintenance:** The motors are maintenance free. They are protected by thermal overloads and will stop automatically if operated repeatedly. Wait until the motors cool before further use.

**Battery Backup:** This lift chair mechanism has a battery backup capability when fitted with 9V battery. In the event of a power failure the battery will return the chair from the reclined position to the sitting position. If this battery backup is used the batteries MUST be replaced as the batteries will power only one operation.